


HOW TO USE ONZETRA® Xsail®


PART 1

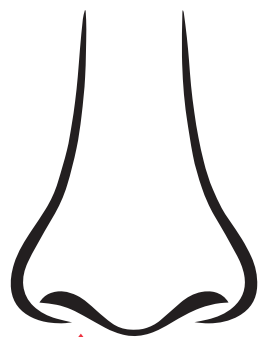

1ST NOSEPIECE
in 1st nostril


- 1** **OPEN** the pouch and remove the **first nosepiece**.

Remove device cover. **INSERT** the first nosepiece into the device until you hear it click into place.

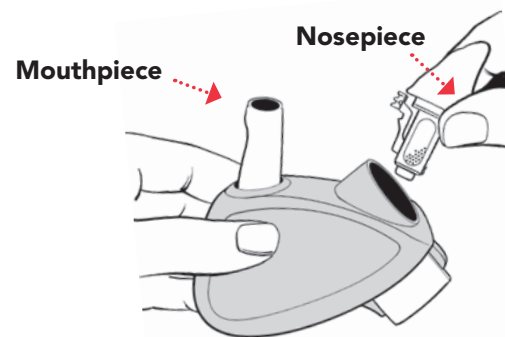


- 2** Fully **PRESS** and **RELEASE** white button to pierce the medication capsule.


! If you skip this step, you will not get any medication

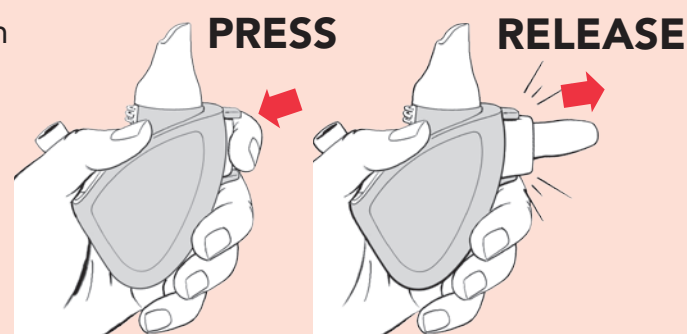


- 3** **INSERT** the nosepiece deeply into your nose. Keep the nosepiece in your nose while you rotate the device to place the mouthpiece into your mouth.

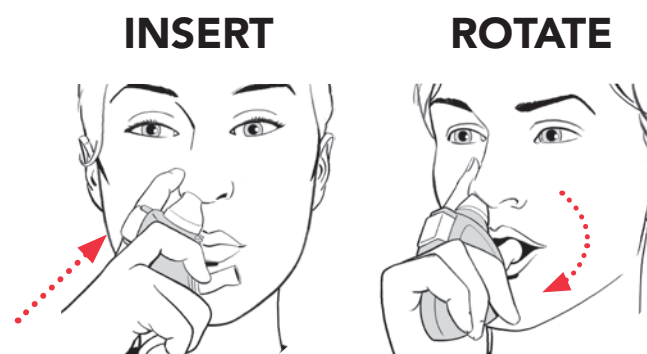

- 4** **BLOW WITH YOUR MOUTH** into the device for **2-3 seconds**, like you're blowing up a balloon, to deliver medication into your nose. (You may hear a rattling sound or feel a vibration.)

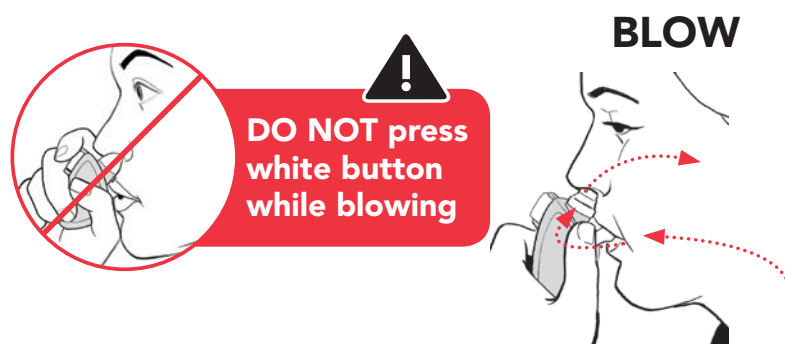


PART 2

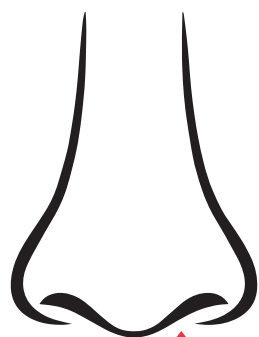


2ND NOSEPIECE
in 2nd nostril

- 5** Press the clear tab to remove the first nosepiece. **CHECK** the capsule to see if the medication is gone. **DISCARD** the used nosepiece in the trash. Insert **second nosepiece** into device.

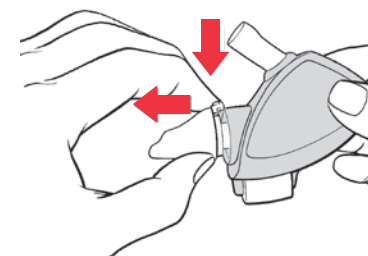

- 6** **DON'T FORGET TO PIERCE THE CAPSULE IN SECOND NOSEPIECE.**

! If you skip this step, you will not get any medication

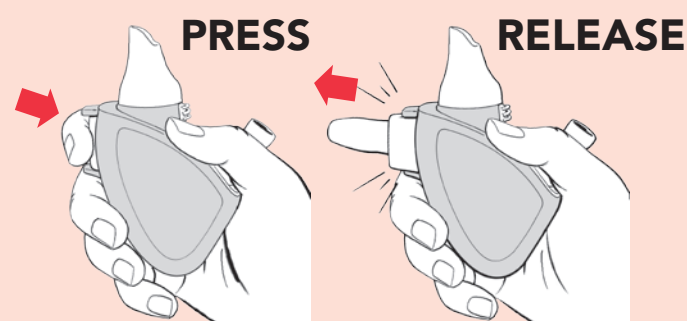

- 7** Lastly, repeat steps 3-4 with **second nosepiece** in **second nostril**.



DID I DO IT RIGHT?

- Checked the nosepieces.** It is normal to see a thin white residue after use. If one or both of the capsules appear unused, repeat steps 2-4 with the same nosepiece
- Used both nosepieces—one in each nostril**
- Pierced both capsules** before blowing
- DID NOT** hold down white button while blowing



These Instructions for Use have been approved by the U.S. Food and Drug Administration.

ONZETRA and Xsail are registered trademarks of Currax™ Pharmaceuticals LLC in the United States and other countries. The other brands listed are trademarks of their respective owners and are not trademarks of Currax™ Pharmaceuticals LLC. The makers of these brands are not affiliated with and do not endorse Currax™ Pharmaceuticals LLC or its products.

U.S. Patent Nos, 6,715,485; 7,975,690; 8,047,202; 8,327,844; 8,550,073; 8,555,877; 8,590,530; 8,875,704; 8,899,229; 8,978,647; 9,108,015; 9,119,932

Distributed by:

Currax™ Pharmaceuticals LLC

Brentwood, TN 37027

1-800-793-2145

Issued January 2024

ONZ-LC103.01

© 2024 Currax™ Pharmaceuticals LLC. All rights reserved